OWANDY-105

COMMUNICATION MADE EASY

WIRED Instruction Manual Rev01


Intraoral Scanner

Instructions, Vejledning,
Gebrauchsanweisung, Instrucciones,
Instructions, Istruzioni, Instructies,
Bruksanvisning, Instruktioner

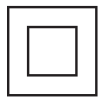
Contents


Labels and Symbols	4	Configuration du scanner	113
EN/English.....	9	Mise en route.....	116
Notice	9	Dépannage.....	120
Overview.....	10	Informations réglementaires	121
Safety information	12	Spécifications techniques	125
Setting up the scanner	19	IT/Italiano	127
Getting started.....	22	Avviso	127
Troubleshooting	25	Panoramica	128
Regulatory information	26	Informazioni sulla sicurezza	130
Technical specification.....	30	Configurazione dello scanner	137
DA/Dansk	32	Informazioni preliminar	140
Meddelelse.....	32	Risoluzione dei problemi	144
Oversigt over	33	Informazioni sulle normative.....	146
Sikkerhedsoplysninger.....	35	Specifiche tecniche.....	150
Opsætning af scanneren	42	NL/Nederlands	152
Kom godt i gang.....	46	Mededeling	152
Fejlfinding.....	49	Overview.....	153
Oplysninger om lovkrav	50	Veiligheidsinformatie.....	156
Tekniske specifikationer.....	54	De scanner installeren.....	163
DE/Deutsch.....	56	Aan de slag.....	166
Achtung	56	Problemen oplossen.....	169
Übersicht	57	Gereguleerde informatie	170
Sicherheitsinformationen.....	59	Technische Specificaties	174
Einrichten des Scanners	66	NO/Norsk	176
Erste Schritte	69	Merknad	176
Fehlerbehebung	73	Oversikt	177
Regulatorische Informationen	74	Sikkerhetsindikasjon	179
Technische Spezifikationen	78	Konfigurer skanneren.....	186
ES/ Español	80	Komme i gang	189
Aviso	80	Feilsøking	192
Descripción	81	Regulatorisk informasjon	193
Información de seguridad	83	Tekniske spesifikasjoner	197
Configuración del escáner.....	90	SV/Svenska	199
Procedimientos iniciales	93	Meddelande	199
Solución de problemas	96	Översikt	200
Información reglamentaria.....	97	Säkerhetsinformation	202
Especificaciones técnicas	101	Konfigurera skannern	209
FR/Français	103	Komma i gång	212
Avis.....	103	Felsökning	215
Aperçu	104	Information om föreskrifter.....	216
Informations de sécurité	106	Tekniska specifikationer	220
		Contact Information	222


Labels and Symbols

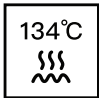	<p>EN CE marking applicable for European Union DA CE-mærkning gældende for EU DE CE-Kennzeichnung gilt für die Europäische Union ES Marca CE aplicable para la Unión Europea FR Marquage CE applicable à l'Union européenne IT Marchio CE applicabile all'Unione Europea NL CE-markering van toepassing voor de Europese Unie NO CE-merking gjelder for EU SV CE-märkning tillämplig för Europeiska unionen</p>
	<p>EN Authorized Representative in the European Community DA Repræsentant i Det Europæiske Fællesskab DE Bevollmächtigter in der Europäischen Gemeinschaft ES Representante autorizado en la Comunidad Europea FR Mandataire Européen IT Rappresentante autorizzato per la Comunità Europea NL Gemachtigde vertegenwoordiger in de Europese Unie NO Autorisert representant i EU SV Auktoriserad återförsäljare inom EU</p>
	<p>EN Importer DA Importør DE Importeur ES Importador FR Importateur IT Importatore NL Importeur NO Importør SV Importör</p>
	<p>EN Refer to instruction manual/booklet DA Se instruktionsbogen/-brochuren. Følg brugsanvisningen DE Siehe Gebrauchsinformation ES Consultar el manual de instrucciones/manual FR Se reporter au manuel/notice d'utilisation IT Consultare le istruzioni del Manuale d'Uso NL Raadpleeg de gebruiksaanwijzing NO Se bruksanvisningen SV Se bruksanvisningen</p>
	<p>EN Electrical insulation Class II DA Elektrisk isolering – klasse II DE Gerät der Schutzklasse II ES Aislamiento eléctrico de clase II FR Isolation électrique Classe II</p>

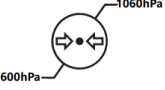	<p>IT Isolamento elettrico Classe II</p> <p>NL Elektrische isolatie Klasse II</p> <p>NO Elektrisk isolasjon klasse II</p> <p>SV Elektrisk isolering klass II</p>
	<p>EN Type BF applied part</p> <p>DA Type BF anvendt del</p> <p>DE Anwendungsteil vom Typ BF</p> <p>ES Componente aplicado de tipo BF</p> <p>FR Partie appliquée de type BF</p> <p>IT Parte applicata di tipo BF</p> <p>NL Type BF Toegepast deel</p> <p>NO Type BF Anvendt del</p> <p>SV Typ BF Tillämpad del</p>
	<p>EN Manufacturer</p> <p>DA Producent</p> <p>DE Hersteller</p> <p>ES Fabricante</p> <p>FR Fabricant</p> <p>IT Fabbricante</p> <p>NL Fabrikant</p> <p>NO Produsent</p> <p>SV Tillverkare</p>
	<p>EN Date of Manufacture</p> <p>DA Produktionsdato</p> <p>DE Herstellungsdatum</p> <p>ES Fecha de manufactura</p> <p>FR Date de fabrication</p> <p>IT Data di Produzione</p> <p>NL Fabricagedatum</p> <p>NO Produksjonsdato</p> <p>SV Tillverkningsdatum</p>
	<p>EN Model Number</p> <p>DA Modelnummer</p> <p>DE Modellnummer</p> <p>ES Número de modelo</p> <p>FR Numéro de modèle</p> <p>IT Numero di modello</p> <p>NL Modelnummer</p> <p>NO Modellnummer</p> <p>SV Modellnummer</p>
	<p>EN Serial number</p> <p>DA Serienummer</p> <p>DE Seriennummer</p> <p>ES Número de serie</p>

	<p>FR Numéro de série IT Numero di serie NL Serienummer NO Serienummer SV Serienummer</p>
	<p>EN Direct current DA Jævnstrøm DE Gleichstrom ES Corriente continua FR Courant continu IT Corrente continua NL Gelijkstroom NO Likestrøm SV Likström</p>
	<p>EN Separate collection for waste electric and electronic equipment (WEEE) is required. DA Der er krav om særskilt indsamling af affald af elektrisk og elektronisk udstyr (WEEE). DE Die getrennte Sammlung von Elektro- und Elektronikaltgeräten (WEEE) ist vorgeschrieben. ES Es necesario recoger por separado los residuos de aparatos eléctricos y electrónicos (RAEE). FR La collecte sélective des déchets d'équipements électriques et électroniques (DEEE) est obligatoire. IT È richiesta la raccolta differenziata dei rifiuti di apparecchiature elettriche ed elettroniche (RAEE). NL Afzonderlijke inzameling van afgedankte elektrische en elektronische apparatuur (AEEA) is verplicht. NO Separat innsamling av avfall fra elektrisk og elektronisk utstyr (WEEE) er påkrevd. SV Separat insamling av avfall av elektrisk och elektronisk utrustning (WEEE) krävs.</p>
	<p>EN Sterilizable in a steam sterilizer (autoclave) at 134°C DA Kan steriliseres i en dampsterilisator (autoklave) ved 134 °C DE Sterilisierbar in einem Dampfsterilisator (Autoklav) bei 134°C ES Esterilizable en esterilizador de vapor (autoclave) a 134 °C FR Stérilisable dans un stérilisateur à vapeur (autoclave) à 134°C IT Sterilizzabile in uno sterilizzatore a vapore (autoclave) a 134°C NL Steriliseerbaar in een stoomsterilisator (autoclaaf) bij 134°C NO Steriliserbar i dampsterilisator (autoklav) ved 134 °C. SV Steriliseras i en ångsterilisator (autoklav) vid 134 °C.</p>
	<p>EN Medical Device DA Medicinsk udstyr DE Medizinisches Gerät ES Productos sanitarios FR Dispositif médical IT Dispositivo medico NL Medisch apparaat NO Medisinsk utstyr SV Medicintekniska produkter</p>

	<p>EN Atmospheric pressure limitation DA Begrænsning af atmosfærisk tryk DE Atmosphärische Druckbegrenzung ES Limitación de la presión atmosférica FR Limitation de la pression atmosphérique IT Limitazione della pressione atmosferica NL Beperking luchtdruk NO Begrensning av atmosfærisk trykk SV Begränsning av atmosfärstryck</p>
	<p>EN Fragile, handle with care DA Skrøbelig, skal håndteres med forsigtighed DE Zerbrechlich, mit Vorsicht zu behandeln ES Frágil, manipular con cuidado FR Fragile, à manipuler avec précaution IT Fragile, da maneggiare con cura NL Breekbaar, voorzichtig behandelen NO Skjør, håndteres med forsiktighet SV Ömtålig, hantera med försiktighet</p>
	<p>EN Humidity limitation DA Begrænsning af luftfugtighed DE Begrenzung der Luftfeuchtigkeit ES Limitación de la humedad FR Limitation de l'humidité IT Limitazione dell'umidità NL Vochtigheidsbeperking NO Begrensning av luftfuktighet SV Begränsning av luftfuktighet</p>
	<p>EN Keep away from rain DA Holdes væk fra regn DE Vom Regen fernhalten ES Mantener alejado de la lluvia FR Tenir à l'écart de la pluie IT Tenere lontano dalla pioggia NL Uit de buurt van regen houden NO Holdes borte fra regn SV Håll borta från regn</p>
	<p>EN Stacking limit by number DA Stablingsgrænse efter antal DE Stapelungsgrenze nach Anzahl ES Límite de apilamiento por número FR Limite d'empilage par nombre IT Limite di impilamento per numero NL Stapellimiet per aantal NO Stablingsgrense etter antall</p>

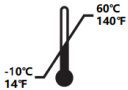	SV Staplingsgräns efter antal
	<p>EN Temperature limits DA Temperaturgrænser DE Temperaturgrenzwerte ES Límites de temperatura FR Limites de température IT Limiti di temperatura NL Temperatuurgrenzen NO Temperaturgrenser SV Temperaturgränser</p>
	<p>EN This way up DA Denne vej op DE Dieser Weg nach oben ES Por aquí arriba FR Par ici IT Da qui in su NL Deze kant op NO Denne veien opp SV Den här vägen upp</p>

EN/English

Notice

We recommend that you thoroughly familiarize yourself with this guide to make the most effective use of your system.

The information in this document is subject to change. Neither Provider nor any of its subsidiaries shall be liable for errors contained herein or for incidental damages in conjunction with the furnishing, performance, or use of this material. No part of this publication may be reproduced without the permission of Provider.

This document is originally written in English.

All trademarks and registered trademarks are the property of their respective holders.

The scanner is intended for professional use only.

U.S. Federal law restricts this device to sale by or on the order of a dentist.

If any serious incident occurs in relation to the device, the user must report it to Provider and to the competent authority of its Member State in the European Union.

Conventions

The following special messages emphasize information or indicate potential risks to personnel or equipment.

	WARNING: Avoids injury to yourself or others by following the safety instructions precisely.
	CAUTION: Alerts you to a condition that might cause serious damage or cause problems.
	NOTE: Provides extra information and hints.


Overview

The scanner is designed to acquire 3D models in the followings:

- Upper jaw
- Lower jaw
- Buccal bite registration


Nomenclature

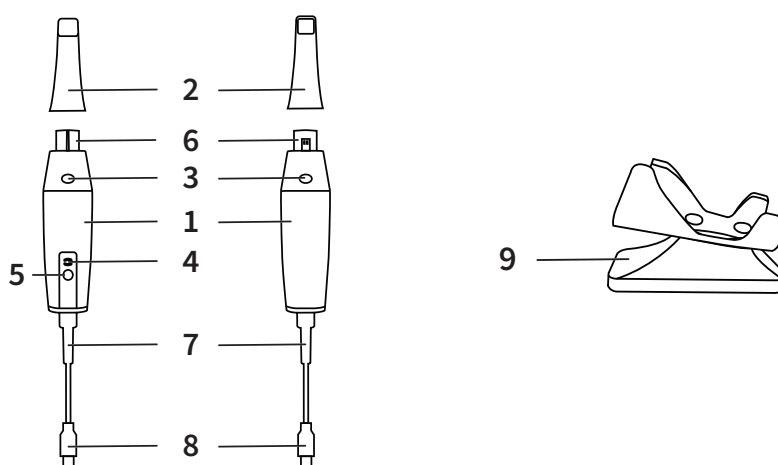The scanner consists of the following parts:

System Composition

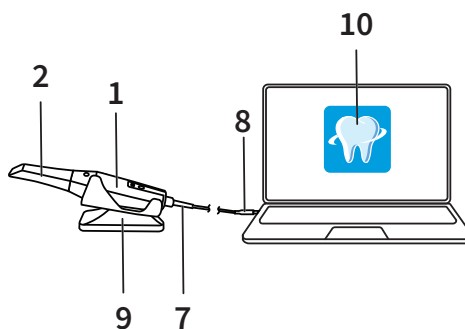Please note that the computer is not included.

[1] Handpiece	
[2] Scanner Tip	
[3] Scan Button	Press to start scanning. Press again to stop scanning.
[4] Mode Indicator	 Upper jaw scan mode  Lower jaw scan mode  Buccal bite registration mode
[5] Mode Switch Button & Status Indicator	Press this button to switch between different modes.
[6] Lens Window	
[7] USB Cable	
[8] USB Connector	
[9] Holder	
[10] Software	

Status Indicator

The relationship between the status of the scanner and the status indicator is as follows:

Scanner Status	Status Indicator	
Power off	Off	
Connected	Solid blue	
Start up	Flashing	
Sleep	Breathing light	

Holder

Place the scanner in the holder when you are not using it.

NOTE: The scanner goes into sleep mode when put into the holder or left idle for 10 seconds. To use it again, take it out of the holder or pick it up and press any button on the scanner.

NOTE: The scanner will automatically shut down if it is idle for more than 10 minutes. If you need to use it again, press any button on the scanner, or take it out of the holder.

Safety information

Intended use

The scanner is a digital optical scanning device used to obtain digital impressions of hard and soft tissues such as teeth, gums, and mucous membrane by means of oral scanning, for oral restoration and orthodontic treatment of malocclusion.

The scanner could be used for both adult and children in clinical practice.

Clinical benefits and performance characteristics

The scanner benefits a dental practice by enabling practitioners to acquire digital impressions with the quality and accuracy required for digital CAD/CAM dental applications. The actual performance of the device is dependent on the user's training and operating execution. The user is solely responsible for the accuracy, completeness, and adequacy of the acquired data.

Contraindications

None.

Warnings and safety instructions

DANGER OF ELECTRIC SHOCK: This is an electrical unit. **DO NOT** expose it to water spray. Such action can cause an electric shock or a malfunction of the unit.

CAUTION: All known residual risks, contraindications, or undesirable side effects are listed in this guide. If any serious incident occurs in relation to the device, you must report it to Provider and to the competent authority of your Member State in the European Union.

WARNING

Scanner

- You **MUST** read and understand this safety information before using the scanner.
- This scanner shall only be used inside hospitals and other professional healthcare facilities and **MUST NOT** be used near high frequency surgical equipment and the RF shielded room of an ME System for magnetic resonance imaging, where the intensity of electromagnetic disturbance is high.

- Before using the scanner, check the outer surfaces of the unit and any accessories to ensure there are no rough surfaces, sharp edges, or protrusions which may cause a safety hazard.
- You are responsible for the operation and maintenance of the scanner. You MUST have training to use the scanner.
- DO NOT place objects within the field of operation of the unit.
- When the unit is not in use, ensure that the scanner is turned OFF.
- DO NOT use the scanner in conjunction with oxygen-rich environments. This unit is not intended for use with flammable anesthetics or flammable agents.
- DO NOT pull or twist the cable.
- DO NOT drop the scanner or the accessories.
- DO NOT sterilize the scanner.
- DO NOT expose the scanner to water spray or submerge it in water or disinfectant.
- DO NOT expose the scanner to high vibrations.
- DO NOT expose the scanner to ultraviolet radiation directly. The scanner is not designed for ultraviolet disinfection.
- DO NOT stare at the LED emission window.
- DO NOT remove the cover of any scanner components. The scanner contains no user-serviceable parts. For any repairs, contact a qualified Provider service technician.
- DO NOT replace the cables provided with the scanner with other cables. Doing so may damage the scanner and adversely affect the safety protection and EMC performance of the scanner.
- Any other equipment not complying with IEC-60601 shall be kept at least 1.5 meters away from the patient.
- If the equipment is faulty, turn it OFF, display an “Out of Service” notice, and contact a qualified Provider service technician.
- Using components, accessories, cables and spare parts other than those specified or provided by the manufacturer of this equipment may impair the safety protection of the scanner and may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- No modification of this equipment is allowed.
- Additional multiple outlet strips or extension cords should not be connected to the system.
- The maximum temperature of the applied part may reach to 43 °C; to avoid overheating, do not use it for extended periods.
- To completely disconnect the device from the main power supply, unplug the USB connector from the USB port.
- DO NOT maintain or service this equipment while it is in use with the patient.
- Connection of the PEMS (Programmable Electrical Medical System) to an IT NETWORK that includes other equipment could result in risks to patients, operators, or third parties. The responsible organization should identify, analyze, evaluate, and control these risks.
- Patients with oral mucosal disease, mental illness, severe respiratory disease, asthma, Parkinson's disease, hyperactivity disease are forbidden.
- Patients with moderate or severe opening limitation should use it with caution.

Computer

- DO NOT place any equipment which does not comply with IEC 60601-1 in the immediate vicinity of the patient. Leave at least 1.5 meters distance between the patient and the equipment.
- The scanner is only intended to be connected to a computer that is at least IEC 60950 / IEC 62368, or equivalent standards certified. Connecting the scanner to other equipment may be hazardous.
- See the installation guide for your computer for information about the data processing system, computer, and screen. Leave enough clear space around the computer to ensure that it is properly ventilated.
- Position the screen to avoid light reflections from internal or external lighting for maximum image quality and visual comfort.

Disposal

This equipment contains certain materials and chemical compounds incidental to the manufacture of electrical and electronic equipment, and improper “end-of-life” disposal of such equipment can result in environmental contamination. Therefore, this equipment should not be disposed of as ordinary household waste but should instead be delivered to a designated electrical and electronic waste disposal or recycling center. For further information on disposing of electrical and electronic waste, contact the cognizant authority within the local jurisdiction. Dispose of the Scanner Tips according to standard operating procedures or local regulations for the disposal of contaminated medical waste. For additional Scanner Tips, contact your dealer.

Cleaning, disinfecting, sterilizing

Perform the following maintenance activities on your scanner and accessories regularly.

To ensure maximum hygienic safety for the patient, carefully follow the instructions to prepare the scanner for use.

To ensure maximum hygienic safety for the patient and to minimize the risk of cross-contamination, carefully perform the following maintenance activities on your scanner and accessories.

After each patient:

- Clean and disinfect the scanner. See “Clean and Disinfect the Scanner”.
- Reprocess the Scanner Tip by applicable method:
 - ◆ Clean the Scanner Tip, and then perform High Level Disinfection (See “Clean the Tip” and “Disinfect the Tip”).
 - ◆ Clean the Scanner Tip, and then perform Autoclave Sterilization (See “Clean the Tip” and “Sterilize the Tip”).

There are 5 models of Scanner Tip as following, applicable to different reprocessing method:

Model	Size	UDI-DI	Manual Cleaning	Automatic Cleaning	High Level Disinfection	134°C Sterilization

TP101	Large	(01)06973993440020	Yes	Yes	Yes	No
TP102	Large	(01)06973993441010	Yes	Yes	No	Yes
TP103	Small	(01)06973993441027	Yes	Yes	No	Yes
TP104	Large	(01)06973993441409	Yes	Yes	No	Yes
TP105	Small	(01)06973993441416	Yes	Yes	No	Yes

Clean and disinfect the scanner

General warnings

WARNING

- Read and follow the warnings and personal protection instructions provided in the Safety Data Sheet (SDS) for the disinfectant used to process the scanner.
- You must wear gloves while cleaning and disinfecting the scanner.
- The scanner must be disinfected with a recommended intermediate-level disinfectant solution with tuberculocidal activity between patients.
- DO NOT use a disinfectant containing phenolics or iodophors; doing so will damage the surface coating of the scanner.
- Never put the scanner in an autoclave device or immerse it in water or the disinfectant solution.
- Excessive fluids can damage the scanner.
- DO NOT use cotton, cloth, or tissues soaked with disinfectant to disinfect the scanner.

Clean the scanner

If the scanner is visibly contaminated with blood and/or body fluids, you must clean it before disinfecting it.

To clean the scanner, follow these steps:

- Dampen (do not soak) a lint-free cloth with lukewarm water.
- Remove the blood and/or body fluids with the dampened lint-free cloth.

Disinfect the scanner

After each patient, the scanner must be thoroughly disinfected.

To adequately disinfect the scanner, follow the disinfectant manufacturer's instructions for the appropriate contact time.

CAUTION: If the scanner is visibly soiled, it must be thoroughly cleaned prior to disinfecting. See “Cleaning the scanner”.

To disinfect the scanner, follow these steps:

- Remove the Scanner Tip.
- Remove all visible soil (See “Cleaning the scanner”).
- Use a commercially prepared intermediate level disinfectant wipe. Follow the manufacturer’s instructions for contact time.

Recommended disinfectant wipes: CaviWipes

WARNING: Using a disinfectant that has not been approved may cause damage to the scanner.

- Thoroughly wipe all surfaces of the scanner. DO NOT allow liquid to enter through the gap, air outlet, or pin holes.

WARNING: Do not rinse.

- Allow to air dry.
- After the scanner has dried, use a clean, lint-free cloth dampened with water to remove residual disinfectant from the surface of the scanner.

Clean the Scanner Tip

WARNING

- Wear gloves when handling a contaminated Scanner Tip.
- Read and follow the warnings and personal protection instructions provided in the manufacturer’s SDS for the detergent used to clean the Scanner Tip.
- DO NOT soak the Scanner Tip in disinfectant for a long period.
- Dry the Scanner Tip thoroughly before mounting onto the scanner.
- DO NOT use an ultrasonic cleaning machine to clean the Scanner Tip.
- Do not soak the Scanner Tip with alcohol-based disinfectants.

Clean the Scanner Tip by one of following ways:

Manually clean the Scanner Tip

To manually cleaning the Scanner Tip, follow these steps:

- Rinse excess soil from the Scanner Tip (2 minutes).
- Using a soft brush, apply enzymatic detergent solution (e.g., Metrex EmPower) to all surfaces.
- Rinse under clean, running water (2 minutes).
- Inspect the Scanner Tip. If it is not clean, repeat the forementioned steps.
- Use a lens tissue or lint-free cloth to remove any dust from the mirror in the Scanner Tip.

Clean the Scanner Tip in an automatic washer

To clean the Scanner Tip in an automatic washer or disinfectant, follow these steps:

- Rinse excess soil from the Scanner Tip (2 minutes).
- Using a soft brush, apply enzymatic detergent solution (e.g., Metrex EmPower) to all surfaces.

-
- Load the Scanner Tip into washer equipment.
 - Run the cycle per the equipment manufacturer's instructions (about 5 minutes). If the machine does not have an automatic rinse cycle, rinse thoroughly to remove detergent residues by immersing in clean water.
 - Use a lens tissue or lint-free cloth to remove any dust from the mirror in the Scanner Tip.
 - Dry the scanner tip's mirror carefully with a lens tissue or lint-free cloth.

Disinfect the Tip (Important for TP101)

To perform high level disinfection after cleaning the Scanner Tip, follow these steps:

- After cleaning the Scanner Tip, check the mirror to make sure the surface is free of stains or fog. Dry the mirror with a lens tissue or lint-free cloth.
- Immerse the Scanner Tip into Johnson & Johnson® Cidex OPA disinfectant for 5 minutes. Detailed disinfection method could be referred to instruction of Cidex OPA.
- After immersing, clean the Scanner Tip according to instruction of disinfectant.
- Use a lens tissue or lint-free cloth to remove any dust from the mirror in the Scanner Tip.

Sterilize the Scanner Tip

Scanner Tip received from the manufacturer is NOT sterilized. You must sterilize them before the first use.

CAUTION: For TP102 and TP103, if you limit the exposure time to no more than 6 minutes at 134°C, you can autoclave the Scanner Tip up to 60 cycles.

CAUTION: For TP104 and TP105, if you limit the exposure time to no more than 6 minutes at 134°C, you can autoclave the Scanner Tip up to 180 cycles.

To perform autoclave sterilization after cleaning the Scanner Tip, follow these steps:

- Put the Scanner Tip into a sealed steam sterilization pouch.
- Place the Scanner Tip in a steam autoclave for sterilization.
 - ◆ Exposure temperature should be set at 134°C.
 - ◆ Exposure time should exceed 3 minutes.
 - ◆ Exposure time should not exceed 6 minutes.

Precautions before use

Perform the following activities on your product and accessories before use.

Visually inspecting the scanner for damage

Visually inspect the scanner for damage or signs of deterioration by doing the following:

- Inspect the scanner's lens window.
- Inspect around the scanner buttons and contact points.

If damage is noted, do not use the scanner, and contact your representative.

Visually inspecting the Scanner Tips

Visually inspect the Scanner Tips for signs of deterioration by doing the following:

- Verify that the Scanner Tip is not damaged, and its components are not detached.
- Verify that the Scanner Tip mirror does not have any smudges or scratches on it.
- If deterioration is noted, replace the Scanner Tip.

WARNING

The lens window on the scanner is a delicate optical component. Mount the front protective cover to protect the lens window from damage and dirt when the scanner is not in use.


The mirror in the Scanner Tip is a delicate optical component. Its clean and undamaged surface is critical to scan quality.

If you see poor scan quality or an unclear video preview in the software, clean the mirror of the Scanner Tip and the Lens Window using a microfiber cleaning swab, applying ethanol that is free of impurities.

Setting up the scanner

To set up the scanner, follow these steps:

- Visit the website to download the software installation package according to the product model (please refer to the download instruction page in the suitcase for website address).
- Double-click the software installation file.
- Choose a language from the drop-down list and click Ok to install.
- Follow the instructions on the screen to complete the installation.
- Firmly slide the Scanner Tip onto the Lens Window.

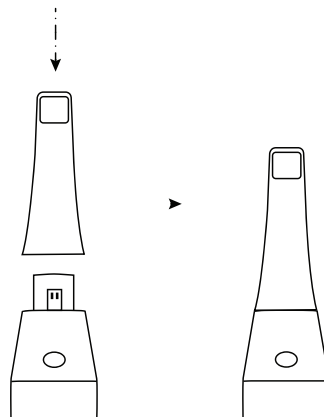- Insert the USB connector of the scanner to any USB 3.0 port on the computer.

CAUTION: Make sure the scanner is connected to the USB 3.0 port. If connected to a USB 2.0 port, the scanner may not work properly.

- The scanner will automatically turn on, the status indicator flashes slowly, and then enters the sleep state, waiting for connection.

NOTE: When the scanner is connected to the software, the indicator stays in solid blue.


Using the holder

The holder can function either as a desktop holder or a wall mount holder.


Installing the desktop holder

To use the holder as a desktop holder, follow these steps:


- Select a clean surface area.
- Place the base of the desktop holder on the desktop and fix the holder bracket on the base with screws.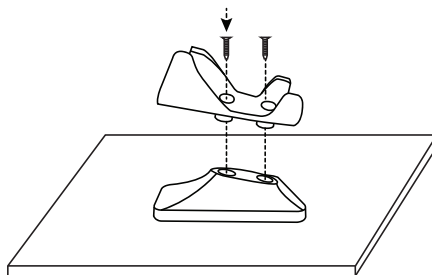

- Choose an appropriate length and fix the USB cable to the end of the base of the desktop holder.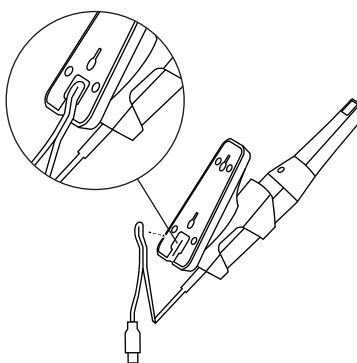

- Place the scanner on the desktop holder as shown.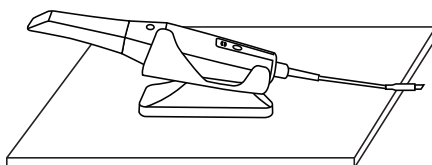


Installing the wall mount holder

To use the holder as a wall mount holder, follow these steps:


Method 1

- Select an area that you can access easily.
- Insert screws through the holes in the holder bracket to affix it to a solid surface.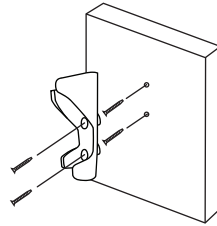


Method 2

- Place the base of the holder on the desktop, and fix the holder bracket on the base with screws.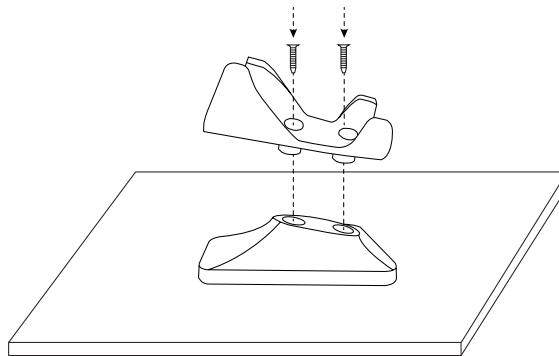

- Select an area that you can access easily.
- Fix the screws to a solid surface.
- Hang the holder to the screws through the holes on the back of the holder base.

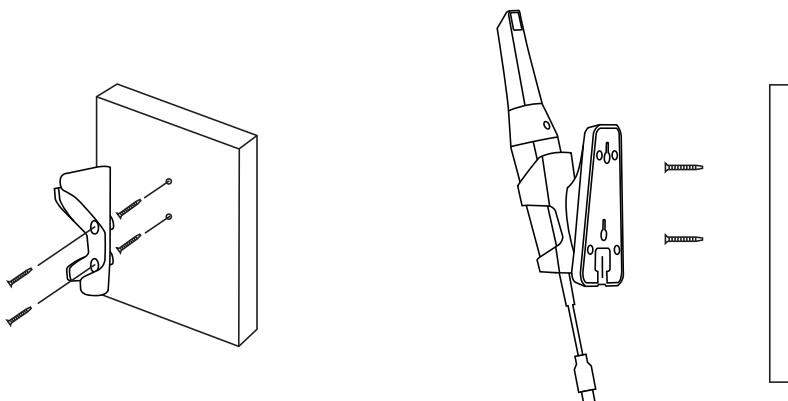NOTE: If the holder is not properly installed, there is a risk that the holder can fall off the wall, resulting in damage to the scanner.

Getting started

Accessing the software user interface

To access the software user interface, follow these steps:

- Double-click the software icon on the desktop.
- The software will automatically pop up the login window.
- If you don't have an account, click **Create an account** to register the organization, and complete the email verification.
- Type your account information in the username and password fields, click the **Log in** button.
- The **Cases** page will be shown up.
- Click the **New Case** button, then input patient name and click the **OK** button.
- Click the **Scan** button.
- If the scanner is not activated, the device activation dialog will be displayed. Follow the instructions on the screen to complete device activation.
- Click the Option menu button  and select **Preferences**.
- Customize the configuration options.
- You can now start acquiring 3D models.

Audio prompts

Scanning sound

When the **Enable scanning sound** option is enabled, your computer will play a continuous sound when you are successfully scanning. If the sound stops, it means the scan has stopped. If you need to continue, please return to the previous scanning area until the scanner resumes scanning and your computer plays a continuous sound. When a bite registration relationship is successfully scanned, your computer will also play a short sound.

Warning sound

If the cumulative scanning time of the current case exceeds the recommended threshold and your computer may not be able to maintain the peak scanning performance, your computer will play a short warning sound (if your computer does not have speakers, this option will not take effect).

NOTE: To enable the audio prompts, your computer must have speakers.

Preparing the teeth

- If there is a preparation area, retract the gingiva by gingival restriction cords. And extract the cords just before scanning the preparation.
- Before starting the scan, dry the teeth thoroughly.
- During the scan, re-dry the teeth moderately.

Preparing the scanner

The Scanner Tip attached to the scanner provides sanitary shield for patients. Always disinfect the body of the scanner and perform high-level disinfection or sterilization on the Scanner Tip after each use.

CAUTION: Scanner Tip received from the manufacturer are NOT sterilized. You must sterilize them before the first use.


CAUTION: Avoid any liquid from leaking into the air outlet near the Scanner Tip mount or the air inlet at the rear of the scanner (see the figure below), otherwise the scanner may be damaged.

To prepare the scanner, follow these steps:

- Make sure the lens window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue.
- Slide the Scanner Tip onto the scanner as shown below.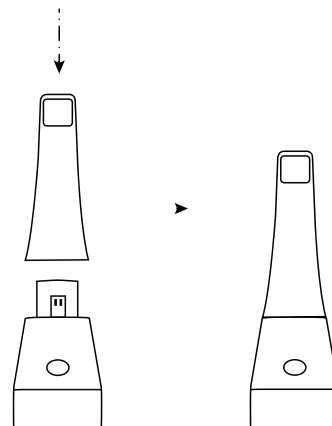


Starting Scanning

To start scanning, place the Scanner Tip on the surface of the tooth to stabilize the scanner and press the Scan button. Wait until a 3D model appears on the 3D model display screen, and then slowly move it along the arch at 0-5mm from the teeth.

Scanning Approach

The recommended scanning method is to start with a molar, since it has greater details for easier identification. Change the scanning angle to less than 60 degrees during scanning to allow the surfaces to overlap, if the overlap is too small, the alignment may be lost.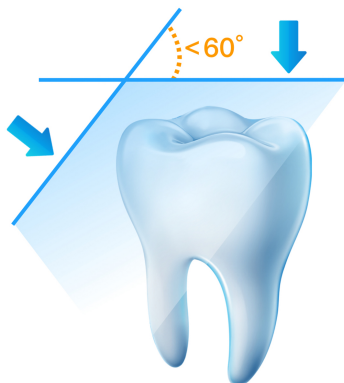


Scanning Protocol

The recommended scanning protocol consisting of 3 sweeps: occlusal, lingual and buccal to ensure good data coverage of all surfaces.

It is recommended to start the first sweep from the occlusal surface, you should start with the first molar.

The second sweep can scan both the lingual and buccal sides, and the third scan covers the opposite side of the second sweep.

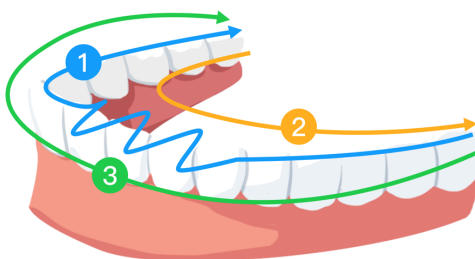Troubleshooting

Scanner troubleshooting instructions

Problem Description	Action
There is mismatching and overlap on the 3D model.	Remove mismatched data and excessive tissue using the Cut tool and rescan.
After bite registration, there is a gap or intersection between the upper jaw and the lower jaw.	<ul style="list-style-type: none"> ● Adjust the "Occlusal pressure level" in Preferences, then disable and enable "Occlusal pressure adjustment". ● Delete the incorrect bite view, and rescan.
Precision degradation is observed, or images are not well-stitched during acquisition.	Ensure that the lens window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue. Use a lens tis-sue or lint-free cloth to remove any dust or water stains. Make sure the Scanner Tip is firmly installed and there are no dark edges on the live video.
Reconstruction of metallic preparations is sometimes difficult.	<p>Adjust the scanner position (for example: distance or angle) and scan more of the area.</p> <p>Move the surgical light away from the patient to decrease light scatter.</p> <p>Turn on "Shining Surface" button.</p>
The Scanner Tip is installed but not detected. No live video is displayed, and the Scanner Tip not detected icon is displayed at the bottom-right of the interface.	Reinstall the Scanner Tip, and make sure the it is in firm contact with the scanner.
Fogging appears on the inner surface of the lens window at the base of the scanner.	Mount a completely dry Scanner Tip on the scanner, and place the scanner in the Holder or set it on the desk, and wait until the fogging fades. If the fogging does not disappear completely after 24 hours, contact your local service provider for assistance. Ensure that the Scanner Tip is thoroughly dry before mounting on the scanner, and do not use a cloth soaked in disinfectant to clean the scanner.
The scanner does not cast light, and a static preview image is displayed in the video preview screen.	Close the software, and then launch the software again.
After stopping scanning, the scanner is disconnected and cannot be automatically reconnected.	<p>Please try the following steps in order.</p> <p>Check if the scanner is turned off. Press any key on the scanner or take it out of the holder to turn on the scanner again. The scanner startup process may take several seconds.</p> <p>Disconnect the USB connector of the scanner from the computer, and then reconnect it to the computer. The scanner will automatically power on and connect.</p>

Regulatory information

The scanner complies with the following regulations:

- (EU) 2017/745 Medical Device Regulation (MDR).
- FDA Center for Devices & Radiological Health CDRH - Title 21 CFR 872.3661 (USA).
- Medical Devices Regulations (Canada).
- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment, 2011/65/EU Annex II and its amendment Directive (EU) 2015/863.

Compliance with European and international standards

EN 60601-1 / IEC 60601-1: Medical Electrical Equipment, Part 1: General Requirements for Basic Safety and Essential Performance

ANSI / AAMI ES 60601-1: Medical Electrical Equipment - Part 1: General requirements for basic safety and essential performance

CAN / CSA-C22.2 No. 60601-1: Medical Electrical Equipment - Part 1: General requirements for basic safety and essential performance

EN 60601-1-2 / IEC 60601-1-2: Medical Electrical Equipment, Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances – Requirements and tests

EN 80601-2-60 / IEC 80601-2-60: Medical electrical equipment — Part 2-60: Particular requirements for the basic safety and essential performance of dental equipment

EN 62471 / IEC 62471: Photobiological safety of lamps and lamp systems: Equipment classification, requirements, and User's Guide

EN ISO 17664: Processing of health care products – Information to be provided by the medical device manufacturer for the processing of medical devices

EN ISO 17665-1: Sterilization of health care products — Moist heat — Part 1: Requirements for the development, validation, and routine control of a sterilization process for medical devices

EN 60601-1-6 / IEC 60601-1-6: Medical Electrical Equipment, Part 1-6: General requirements for basic safety and essential performance - Collateral Standard: Usability

EN 62366-1/ IEC 62366-1: Medical devices - Application of usability engineering to medical devices

EN 62304 / IEC 62304: Medical device software - Software life cycle Processes

EN ISO 10993: Biological evaluation of medical devices

EN ISO 14971: Medical devices - Application of risk management to medical devices

EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements

EN ISO 20417: Medical devices — Information to be supplied by the manufacturer

ISO 9687: Dentistry - Graphical symbols for dental equipment

AAMI TIR 12: Designing, testing, and labeling reusable medical devices for reprocessing in health care facilities: A guide for medical device manufacturers

AAMI TIR 30: A compendium of processes, materials, test methods, and acceptance criteria for cleaning reusable medical devices

Classification in accordance with EN / IEC 60601-1

Type of protection against electric shock: Class II

Degree of protection against electric shock: Type BF Applied Part

Mode of operation: Continuous operation

Flammable anesthetics: Not suitable for use in the presence of flammable anesthetics or a mixture of flammable anesthetics with air or oxygen or nitrous oxide.

Conformity with EN/ IEC 60601-1-2

IEC 60601-1-2: EMC requirements and tests, Medical Electrical Equipment including CISPR 11 Group 1, Class B.

Electromagnetic Compatibility Precautions

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC).

Medical equipment must be installed and put into service according to the EMC information provided in this documentation.

Other equipment can interfere with communications with the scanner, even if the equipment complies with CISPR emissions requirements.

WARNING: Portable and mobile RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the scanner, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Guidance and manufacturer's declarations

Guidance and manufacturer's declaration - electromagnetic emissions

The scanner is intended for use in the electromagnetic environment specified below. The customer or user of the scanner should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1 Class B	The scanner uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

Guidance and Manufacturer's declaration – Electromagnetic immunity for equipment and systems

The scanner is intended for use in the electromagnetic environment specified below. The customer or the user of the scanner should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic discharge (ESD) IEC	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with

61000-4-2			synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Radiated RF IEC 61000-4-3	3 V/m 80MHz – 2.7GHz	3 V/m 80MHz – 2.7GHz	Environment of a professional healthcare facility. WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the scanner including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

For the immunity to proximity fields from RF wireless communications equipment, the scanner is compliant with the test levels specified below, according to IEC 60601-1-2 standard. The customer or user of the scanner should assure that it is used in such an environment.

Test Frequency (MHz)	Band (MHz)	Immunity Test Levels
385	380 – 390	Pulse modulation 18Hz, 27V/m
450	430 – 470	FM, ±5 kHz deviation, 1 kHz sine, 28V/m
710	704 – 787	Pulse modulation 217Hz, 9V/m
745		
780		
810	800 – 960	Pulse modulation 18Hz, 28V/m
870		
930		
1720	1700 – 1990	Pulse modulation 217Hz, 28V/m
1845		
1970		
2450	2400 – 2570	Pulse modulation 217Hz, 28V/m
5240	5100 – 5800	Pulse modulation 217Hz, 9V/m
5500		
5785		

Accessories

The use of cables or accessories other than those specified, with the exception of those sold by the manufacturer of the equipment, as replacement parts for internal components may result in increased emissions or decreased immunity of the medical equipment.

Other equipment

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify normal operation.

Technical specification

Model

OWANDY-IOS

Scanner technical specifications

Components	Technical Specifications
Weight	Scanner (cable included) without Scanner Tip: 260g Tip Large – High Level Disinfection (TP101): 12g Tip Large – Sterilization (TP102): 14g Tip Small – Sterilization (TP103): 12g Tip Large – Sterilization (TP104): 14g Tip Small – Sterilization (TP105): 12g
Color	3D full color
Connectivity	USB 3.0
fd	USB 3.0 5V, 900mA
Field of view	Tip Large – High Level Disinfection (TP101): 16mm x 14mm Tip Large – Sterilization (TP102): 16mm x 14mm Tip Small – Sterilization (TP103): 12mm x 12mm Tip Large – Sterilization (TP104): 16mm x 14mm Tip Small – Sterilization (TP105): 12mm x 12mm
Minimum system requirements of laptop	Processor: 10th Generation Intel® Core™ i7-10750H or AMD Ryzen™ 7 4700U Memory: 16 GB Graphics card: NVIDIA GeForce GTX 1650 4GB Storage: 512 GB SSD Operating system: Win 10 (build 18362+) / Win 11, 64 bit Others: USB 3.0 port Optional: Touch screen
Minimum system requirements of desktop	Processor: 9th Generation Intel® Core™ i5-9600 or AMD Ryzen™ 5 2600 Memory: 16 GB Graphics card: NVIDIA GeForce GTX 1650 4GB Storage: 512 GB SSD Operating system: Win 10 (build 18362+) / Win 11, 64 bit Others: USB 3.0 port Optional: Touch screen

CAUTION: It is MANDATORY to check that your system configuration is compatible with the computer system requirements for the scanner software.

Environmental Requirements

Components	Environmental Requirements
Operating temperature	15°C ~ 30°C
Transport and storage temperature	-10°C ~ 60°C
Operating relative humidity	10% ~ 65% RH
Transportation and storage relative humidity	10% ~ 95% RH
Operating atmospheric pressure	70 ~ 106 KPa
Transportation and storage atmospheric pressure	60 ~ 106 KPa

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