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INSTRUCTION MANUAL ○●○

BRUSHLESS MICRO MOTOR

OZ
OZ-plus
OZ-black
OZ-black-D

SAESHIN

1 English / 한국어

OZ

OZ-plus / FORTE100αIII

OZ-black / FORTE100αIII

OZ-black-D / FORTE400αI

English / 한국어

* Read all instructions before using. This product is mainly designed for dental laboratory use.
(It can be applied for the nail art, jewelry, wood carving and molding use.)

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Proper maintenance and repair will bring long life to this product.

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Accidents that occur by following the instruction for use manual is not Saeshin's responsibility.

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Products' appearance or specification is subject to change without notice.

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The user is responsible for the operation and maintenance of this product.

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When informing the model, serial number, breakage condition and contact detail will be necessary in order for Saeshin to provide consultation.

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Saeshin warrants our products for manufacturer's fault such as material or workmanship for one year from the date of invoice. The warranty does not cover the product fault caused by user's misuse, wrong installation, inappropriate maintenance and repair, and normal wear of consumable parts such as bearings, spindle case assy, armature etc. In order to check the warranty, it's required to provide the condition and environment used in information with product serial number and stamped invoice. The warranty is by manufacturer's product analysis and at its discretion to repair or replace the product

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Exception of warranty

user's misuse or improper use and treatment
using with wrong input Voltage (AC Voltage)
dropping down during setting, moving or using
using consumable parts or accessories not recommended
malfunction after repairing at the places not authorized or recommended by manufacturer
replacement of normal wear of consumable parts (ball bearings, gear assy, collet chuck, armature etc) An Act of God

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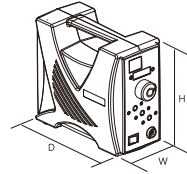
* Design and specification of the product can be subject to be changed without notice for improving the quality.

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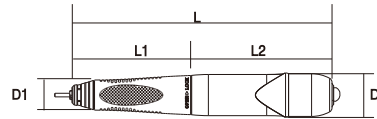
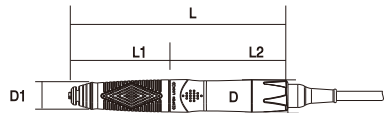
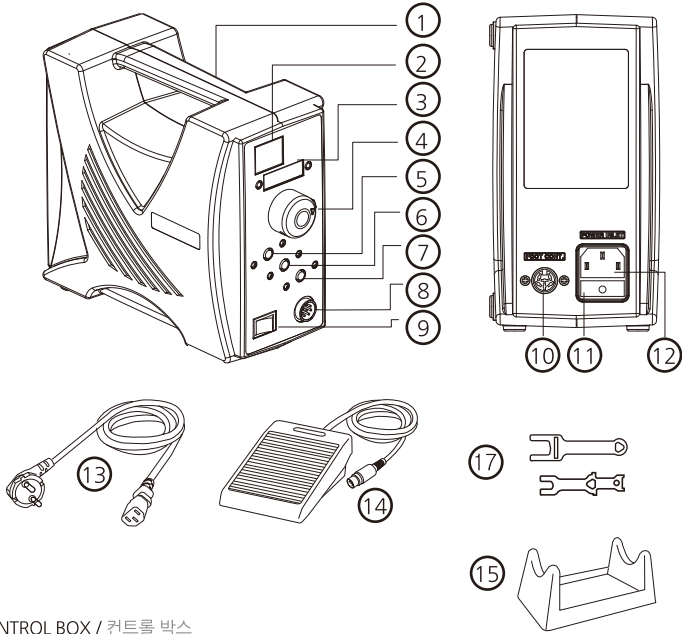
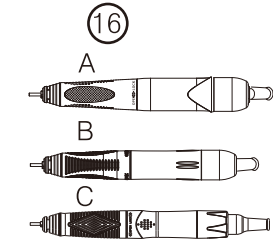
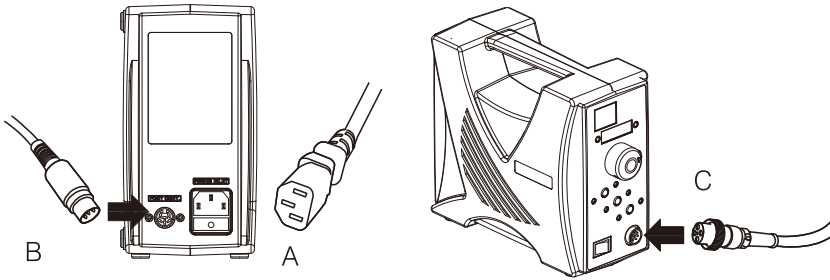
Please make sure that power is off when connecting the motor and foot pedal.
(Otherwise, it might cause malfunction.)

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POWER ON (I)

Turn on the power switch.
With beep sound, the power switch lights green and shows a number on the rpm display box.

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POWER OFF (O)

The green light on the power switch and the lights of the buttons go off.
The numbers on the rpm display box is disappears.

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40 ↔ 60
50

9 OZ : How to change the speed from 40,000rpm to 60,000rpm
OZ-plus / OZ-black / OZ-black-D : 40,000rpm to 50,000rpm
Put the volume dial at '0' and then press the hand/foot button and motor button at the same time. The RPM display will be changed from 50,000 rpm or 60,000 rpm. After repeating, the RPM display will be returned to 40,000rpm. The locking system which the RPM functions is variable.
OZ : 40,000rpm에서 60,000rpm 으로 변환방법
OZ-plus / OZ-black / OZ-black-D : 40,000 rpm 50,000 rpm 불륨다이얼을 0으로 돌린 후 손/발 버튼과, MOTOR 버튼을 같이 눌러줍니다.
RPM표시창이 50,000 또는 60,000으로 변경됩니다.
다시 한번 더 반복하면 원래대로 40,000으로 변경됩니다.
락에서의 RPM수치는 일정하지 않게 됩니다.

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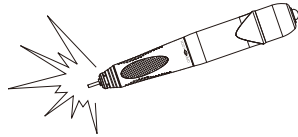
In case of overload, after 5 seconds, control box stops and shows Error 1 with beep sound.
To release, push motor button when working by hand or take off foot from the pedal in case of working by pedal.

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1. Keep the handpiece surroundings clean.

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2. Remove any dust on handpiece with a smooth clothe with isopropyl alcohol and avoid getting wet inside.

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■ Do not apply oil or water to the unit when cleaning.

as this may cause the bearings to be damaged or cause corrosion of the inner-parts.

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3. Air cleaning is acceptable, but air pressure may negatively affect the handpiece.

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4. insertion and pulling out the bur

Open the chuck turning handle in an open direction. Chuck is released and bur can be removed.

Turn the handle in a lock direction until you hear a click sound. Chuck is locked and bur is in the correct position.

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WARNING / 경고

When turning in a lock direction too strongly the headpiece handle and motor part may become disassembled.
Do not disassemble except by a skilled person as it may cause damage to the unit.

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CAUTIONS / 주의

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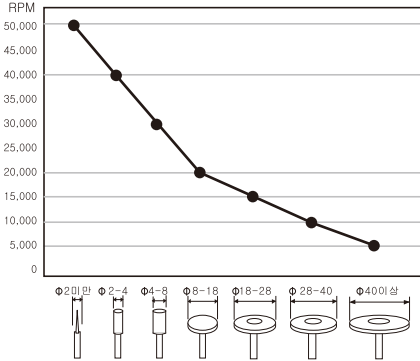
[The allowable RPM according to the bur diameter]

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Please keep the RPM according to the diagram below to protect against an accident.

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The allowable RPM could be different from
the bur material and type.

* Please refer the speed suggested by the bur
manufacturer.

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※ When chuck is locked, the maximum projection length
of bur is more than 35mm.

Use the bur ISO standard and avoid using a rusty or bent
bur which can cause excessive wear of the bearings.
Keep a safe speed and precautions recommended by the
bur manufacturer.

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2. Be careful not to drop the handpiece as it may be
damaged or broken. In case that it is dropped, check if
the bur is bent or that any excessive heat is generated
while carefully operating it.

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3. If any problem occurs in handpiece, only use after
repairing.

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4. Make sure that the switch is off before putting the
power plug in the socket, in order to avoid unexpected
injury caused by a sudden start.

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5. Circuit breaker is provided to protect the motor.

It is actuated when an overload occurs which the motor
cannot stand.

Always use this product within allowable load range.

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6. Do not open the chuck while operating.

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7. Keep a bit-bur in the collet chuck of the handpiece
after using.

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* If the handpiece is kept without bur or bit bur,
the collet chuck may shrink from the environment condition
and may affect some spare parts.

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8. Do not put the handpiece down before it stops
completely.

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9. Be sure to pull out the power cord before detaching
the control box.

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11. Use only accessories suitable for this product.

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12. Follow the manufacturer's recommended
instructions for bur speeds for various applications.

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13. Wear proper protective clothes and equipment,
including a dust mask and safety goggles. Use a proper
suction unit and a ventilator.

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14. Keep the workplace clean as debris around the
workplace can cause an accident.

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CAUTIONS / 주의

15. Memory / 메모리

Memory saves the conditions of direction and hand/foot selection when turning the power off.

Therefore, be careful when turning the power on.

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16. Do not work in a dangerous place.

Do not use the control box in a humid place.

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17. The handpiece and foot pedal can only be used with a Saeshin control box.

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18. Do not touch the signal input, signal output or other connectors and the patient simultaneously.

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19. External equipment intended for connection to signal input, signal output or other connectors, shall comply with relevant IEC Standard (e.g., IEC60950 for IT equipment and IEC60601-1 series for medical electrical equipment).

In addition, all such combination-system shall comply with the standard IEC60601-1 and/or IEC60601-1-1 harmonized national standard or the combination.

If in doubt, contact a qualified technician or dealer.

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20. Check for any troubles, including vibration of bur, at low speeds, and at high speeds.

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21. Operate the speed control dial of the control box slowly.

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22. Turn the power off or set the RPM at zero and to change a bur.

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23. Do not turn the power on in control box when the collet chuck of handpiece is open. (E1 error)

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* CHANGING FUSE / 퓨즈 교체

Check the fuse if the control box doesn't work.
It may be necessary to change it.
Contact the dealer or the manufacturer for more information.

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This marking shown on the product or its literature, indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract.

This product should not be mixed with other commercial wastes for disposal.

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Serial Number

WEEE marking

Caution, consult accompanying documents

Alternating current

Mains switch On

Fuse

Mains switch Off

EC Certificate Full Quality Assurance System

Do not use with a damaged or faulty power cord, plug, and outlet.

It may cause danger of fire and/or electric shock. Contact the manufacturer.

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When unplugging the power cord, do not touch with wet hand.

Do not use with a loose power plug which can cause danger of electric shock and/or fire.

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Do not bend the power cord excessively and do not press-down with a heavy object as this may cause electric shock and/or fire.

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Do not place near of heated equipment, candles or cigarettes which can cause a fire.

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Do not leave children unattended in any room where this product has been installed.

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- Type of protection against electric shock : Class I equipment
- Degree of protection against electric shock: Type B applied parts
- Classification according to the degree of protection against ingress of water as detailed in the current edition of IEC 60529: Equipment: IPX0, ordinary equipment
Foot Switch: IPX1
- This equipment is not suitable for use in the presence of flammable anesthetics or oxygen
- Mode of operation: continuous operation

"This equipment has been tested and found to comply with the limits for medical devices in EN 60601-1-2:2007.

These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

This equipment can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Re-orient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a different circuit from that to which the other device(s) are connected.
- Consult the manufacturer or dealer for help."

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Use only a grounded conductor.

Breakage or electric leak may cause danger of electric shock. For grounding, inquire with an electrician or manufacturer.

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In case of using extension cord, do not use many electrical equipment at one time.

As it may cause fire and/or electric shock.

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Do not place or set up the product in a humid, rainy or snowy location.

As it may cause the danger of electric shock and/or fire.

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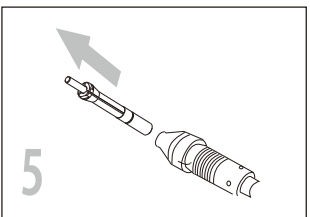
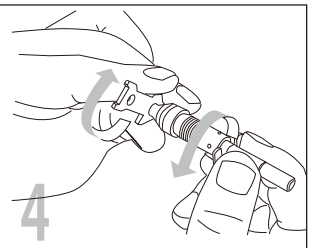
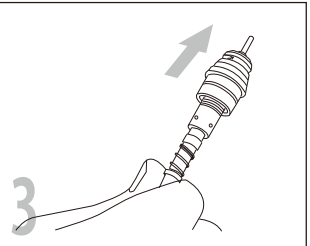
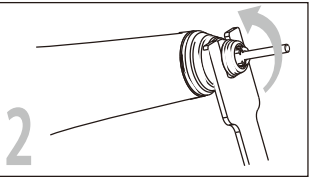
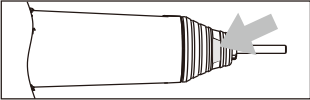
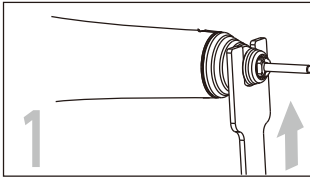
Do not drop or damage the handpiece.

In case of malfunction, dropping, drowning, do not use and contact the manufacturer. (If may cause vibration of the bur, overheating, or damage to the bearings.)

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* Use the products in properly as per the instruction manual and do not use products not recommended by the manufacturer as it may cause injury or property loss
Saeshin is not responsible for any accident caused by not following the instruction manual.

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Attention / 주의사항

Clean and assemble the chuck very carefully.
If not skilled person, do not disassemble the handle and motor part as it may cause the unit to break.
If disassembled, delink of the joint of handle part and the shaft of the motor part have to be fit together, otherwise vibration, noise, heat and bearing damage may occur.

*If the chuck is not open, please turn it with the wrench.

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SAESHIN PRECISION CO., LTD.